

○

**VOLUME**  
2 beeps (remove for 2 sec)  
1 beep (hold)

Volume increases in 1 step increments and continues until SoundLync is removed

Maximum setting: double beep

**SoundLync™ Quick Reference**    Phonak Lyric™

**VOLUME**  
2 beeps (remove for 2 sec)  
1 beep (hold)

---

Volume will start to increase in 1 step increments and continue until SoundLync is removed at desired volume

---

Maximum setting: double beep

**PHONAK**  
life is on

**SoundLync™ Quick Reference**    Phonak Lyric™

**ON** 1 beep (remove)


---

**SLEEP** 2 beeps (remove)

---

**OFF** 2 beeps (hold)  
4 beeps

028-1223-02/V3.00/2021-09/hc © 2021 Sonova AG. All rights reserved.



○

**Phonak Lyric™**

---

**SoundLync™**  
ON 1 beep (remove)  
SLEEP 2 beeps (remove)  
OFF 2 beeps (hold)  
4 beeps

**PHONAK**

**PHONAK**  
life is on