

Phonak Terra™ Phonak Terra™ + RIC ●

User Guide

Phonak Terra RIC-312
Phonak Terra+ RIC-312




Your hearing aid details

This user guide is valid for:

Wireless hearing aids

Phonak Terra RIC-312
Phonak Terra+ RIC-312



 If no box is checked and you do not know the model of your hearing aid, please ask your hearing care professional.

Hearing aid models

- Terra RIC-312
- Terra+ RIC-312

Battery size

- 312
- 312

Earpieces

- Dome
- SlimTip
- cShell

Wax filter tools

- CeruStop
- CeruShield™



Contents

Your hearing aids have been developed by Phonak – a world leader in hearing solutions based in Zurich, Switzerland.

These premium products are the result of decades of research and expertise and are designed with your well-being in mind. We thank you for making such a great choice and wish you many years of listening pleasure. Please read the user guide carefully to make sure that you understand and get the best out of your device. Training is not required for handling of this device. A hearing care professional will help set up this device according to your individual preferences during the fitting consultation. For more information regarding features, benefits, set up, use, maintenance or repairs of your hearing aid and accessories, please contact your hearing care professional or the manufacturer representative. Additional information can be found in the datasheet of your product.


Phonak - life is on
www.phonak.com

Your hearing aid		
1.	Quick guide	7
2.	Parts of the hearing aid	9
Instructions for using the hearing aid		
3.	Left and right hearing aid markings	10
4.	On/Off	11
5.	Batteries	12
6.	Putting on the hearing aid	14
7.	Removing the hearing aid	15
8.	Multi-function button	16
9.	Connectivity overview	17
10.	Initial pairing	18
11.	Phone calls	21
12.	Flight mode	26


Further information	
13. myPhonak App overview*	28
14. Environmental conditions	29
15. Care and maintenance	31
16. Exchanging the earwax protection	34
17. Service and warranty	41
18. Compliance information	43
19. Information and description of symbols	48
20. Troubleshooting	54
21. Important safety information	56

1. Quick guide


Changing batteries

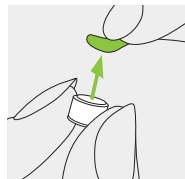

1.
Remove the sticker from the new battery and wait 2 minutes.

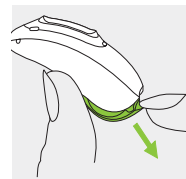


2.
Open the battery door.

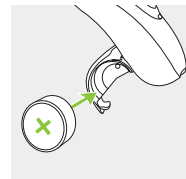


3.
Place battery in the battery door with the flat side facing upwards.

Left and right hearing aid markings

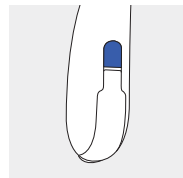


Blue marking for left hearing aid.

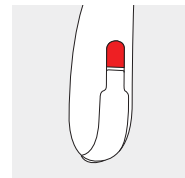


Red marking for right hearing aid.

On/Off

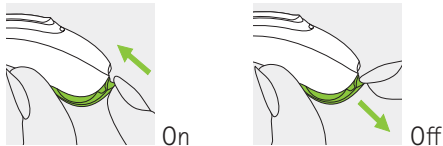


Multi-function button

The button has several functions. It functions as a volume control and/or a program change depending on the hearing aid programming. This is indicated in your individual "Hearing aid instructions". If paired with a Bluetooth® enabled phone, a short press will accept and a long press will reject an incoming call.

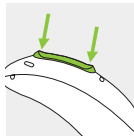

Flight mode

To enter flight mode, press the lower part of the button for 7 seconds while closing the battery door. To exit flight mode, simply open and close the battery door again.

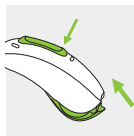


2. Parts of the hearing aid

The following pictures show the hearing aid model and compatible earpieces described in this user guide.


You can identify your personal model by:

- Checking "Your hearing aid details" on page 3.
- Comparing your earpiece and your hearing aid with the following models


Compatible earpieces

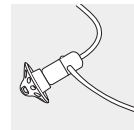

Dome

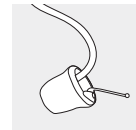


SlimTip

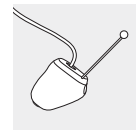




cShell

Terra RIC-312 / Terra+ RIC-312

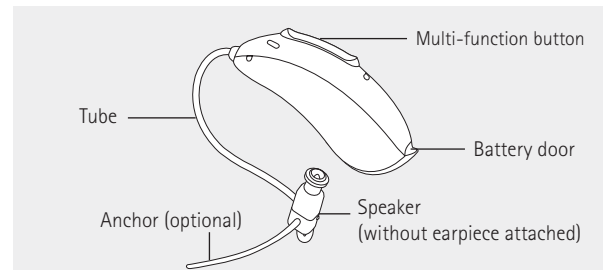


3. Left and right hearing aid markings

There is a red or blue marking on the back of the hearing aid and on the speaker. This will tell you if it is a left or right hearing aid.

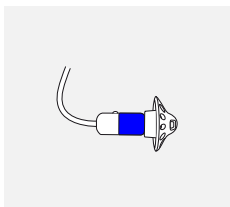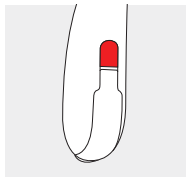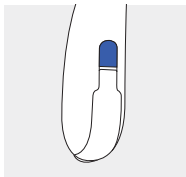

Blue marking for left hearing aid.

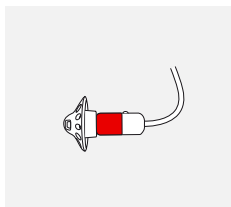


Red marking for right hearing aid.


4. On/Off

The battery door is also the on/off switch.

1.
Closed battery door =
hearing aid is **on**

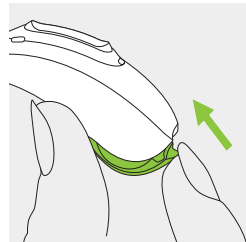


2.
Open battery door =
hearing aid is **off**

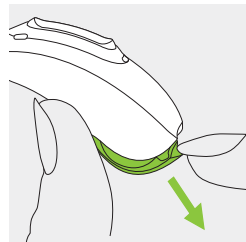


① When you switch on the hearing aid you may hear a start-up melody.


5. Batteries

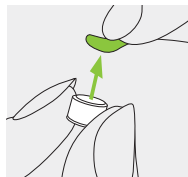

1.
Remove the sticker from the new battery and wait 2 minutes.

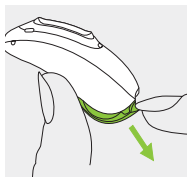


2.
Open the battery door.

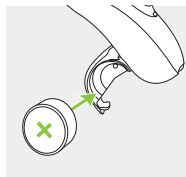

3.
Place battery in the battery door with the flat side facing upwards.

❗ If it is difficult to close the battery door: check that the battery is inserted correctly and that the flat side is facing upwards. If the battery is not inserted correctly, the hearing aid will not work and the battery door can be damaged. The flat side is marked with a "+" symbol either on the sticker or on the battery.

 **Low power:** You will hear two beeps when the battery is low. You will have approximately 30 minutes to change the battery (this can vary, depending on the hearing aid settings and battery). We recommend that you always have a new battery on hand.

Replacement battery


This hearing aid requires a size 312 zinc-air battery.

Model	Zinc-air battery size	Color marking on package	IEC code	ANSI code
Phonak Terra RIC-312				
Phonak Terra+ RIC-312	312	brown	PR41	7002ZD


❗ Please ensure that you use the correct type of battery in your hearing aid (zinc-air). Please also read chapter 21.2 for further information on product safety.

6. Putting on the hearing aid


1.
Place the hearing aid behind your ear.

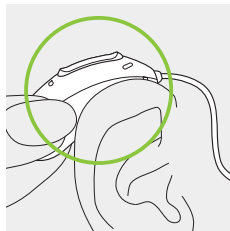

2.
Insert the earpiece into your ear canal.

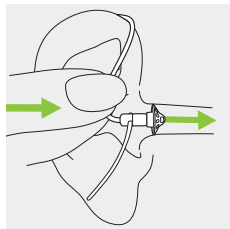


3.
If there is an anchor attached to the earpiece, tuck it into the bowl of your ear to secure your hearing aid.

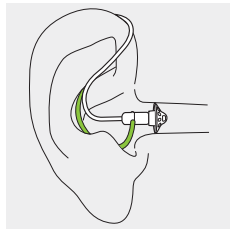

7. Removing the hearing aid


1.
Pull on the bend of the tube and remove the hearing aid from behind the ear.

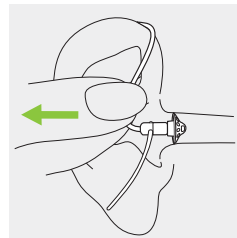

8. Multi-function button

The multi-function button has several functions.


It functions as a volume control and/or a program change depending on the hearing aid programming. This is indicated in your individual "Hearing aid instructions". Please ask your hearing care professional for a printout.

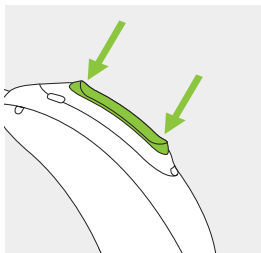

If the hearing aid is paired with a Bluetooth® enabled phone, a short press on the upper or the lower part of the button will accept an incoming call and a long press will reject an incoming call – refer to chapter 11.

9. Connectivity overview

The illustration below shows the connectivity options available for your hearing aid.

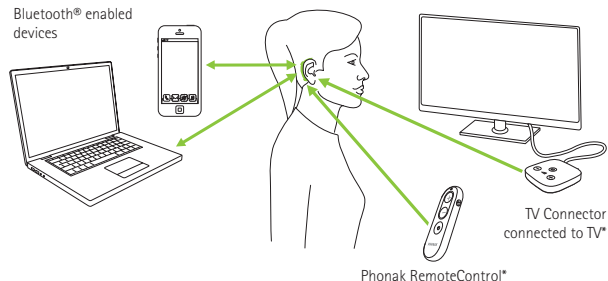

The TV Connector can be connected to any audio source such as a TV, PC or hi-fi system.

* Available only for Terra+ RIC-312


10. Initial pairing

10.1 Initial pairing to a Bluetooth® enabled device

① It is only necessary to perform the pairing procedure once with each device featuring Bluetooth® wireless technology. After the initial pairing, your hearing aid will connect automatically to the device. Initial pairing can take up to 2 minutes.

1.
On your device (e.g. a phone), ensure that Bluetooth® wireless technology is enabled and search for Bluetooth® enabled devices in the connectivity setting menu.

2.
Switch on both hearing aids.
You now have 3 minutes to pair your hearing aids with your device.

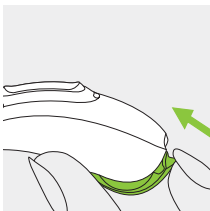

3.
Your device shows a list of Bluetooth® enabled devices. Select the hearing aid from the list to simultaneously pair both hearing aids. A beep confirms successful pairing.

① For more information about pairing instructions for Bluetooth® wireless technology, specific to some of the most popular phone manufacturers, go to: <https://www.phonak.com/com/en/support.html>


10.2 Connecting to the device

After being paired to your device, your hearing aid will automatically connect again when switched on.

- ① The connection will be maintained as long as the device remains ON and within range.
- ① Your hearing aid can be connected to up to two devices and paired to up to eight devices.

11. Phone calls

Your hearing aid connects directly with Bluetooth® enabled phones. When paired and connected to your phone, you will hear notifications and the caller's voice directly in your hearing aid. Phone calls are "hands free", this means that your voice is picked up by the hearing aid microphones and transmitted back to the phone.

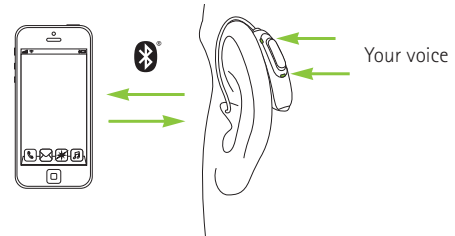


11.1 Making a call

Enter the phone number and press the dial button. You will hear the dialing tone through your hearing aid. Your voice is picked up by the hearing aid microphones and transmitted to the phone.

11.2 Accepting a call

When receiving a call, an incoming-call notification will be heard in the hearing aid.

The call can be accepted by a short press on the upper or lower part of the multi-function button on the hearing aid (less than 2 seconds) or directly on your phone.

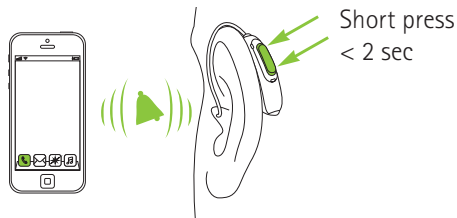


11.3 Ending a call

A call can be ended by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone.



11.4 Rejecting a call

An incoming call can be rejected by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone.

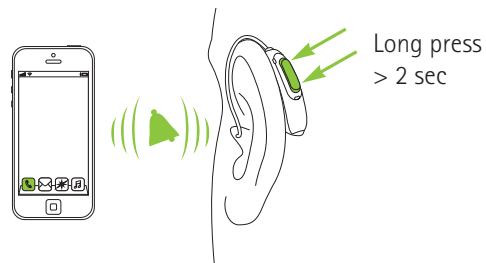


12. Flight mode

Your hearing aid operates between 2.4 GHz–2.48 GHz frequency range. When flying some operators require all devices to be switched into flight mode. Entering flight mode will not disable normal hearing aid functionality, only the Bluetooth® connectivity functions.


12.1 Enter flight mode

To disable the wireless function and enter flight mode in each hearing aid:

1.
Open the battery door.

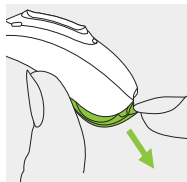

2.
Hold down the lower part of the multi-function button on the hearing aid for 7 seconds while closing the battery door.

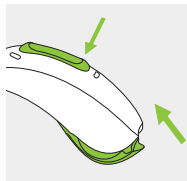


In flight mode, your hearing aid cannot connect directly to your phone.


12.2 Exit flight mode

To enable the wireless function and exit flight mode in each hearing aid:

1.
Open the battery door.

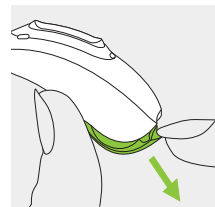

2.
Close the battery door again.

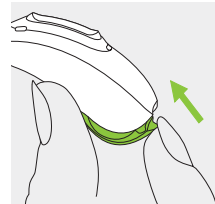

13. myPhonak App overview*

To learn more about the myPhonak app, please visit the webpage www.phonak.com/myphonakapp or scan the QR code.



To install the myPhonak app, please scan the QR code.



* Available only for Terra+ RIC-312.

14. Environmental conditions

The product is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in this user guide.

Please ensure to use, transport and store the hearing aid according to the following conditions:

	Operating	Transport	Storage
Temperature	+5 to +40 °C (+41 to +104 °F)	-20 to +60 °C (-4 to +140 °F)	-20 to +60 °C -4 to +140 °F
Humidity (non condensing)	0% to 85%	0% to 93%	0% to 93%
Atmospheric pressure	500 to 1060 hPa	500 to 1060 hPa	500 to 1060 hPa

Temperature and humidity shall not exceed the ranges described in the table above for extended periods of time during transport and storage.

Remove the battery if you are not using your hearing aid for a long period of time. Prolonged storage at temperatures lower than 10°C and higher than 30°C may negatively affect the battery performance of the product.



15. Care and maintenance

These hearing aids are classified as IP68. This means that they are water and dust resistant and designed to withstand daily life situations. They can be worn in the rain but should not be fully submerged in water or used when taking a shower, swimming or other water activities. These hearing aids should never be exposed to chlorinated water, soap, salt water or other liquids with a chemical content.

If the device has been subject to storage and transport conditions outside of the recommended use conditions, wait 15 minutes before starting the device after recommended environmental conditions are restored.

Expected Service Life:

The hearing aids have the expected service life of five years. The devices are expected to remain safe for use for this duration.

Commercial service period:

Routine and diligent care of your hearing aids will contribute to the outstanding performance for the duration of the expected service life of your devices. Sonova AG will provide a minimum of a five year repair service period after the respective hearing aid and essential components have been phased out of the product portfolio.

Please use the following specifications as a guideline. For further information regarding product safety, see chapter 21.2.

General information

Before using hair spray or applying cosmetics, you should remove your hearing aid from your ear, because these products may damage it.

When you are not using your hearing aid, leave the battery door open so that any moisture can evaporate. Make sure that you always completely dry your hearing aid after use. Store the hearing aid in a safe, dry and clean place.

Your hearing aid is resistant to water, sweat and dust under the following conditions:

- The battery door is fully closed. Ensure that no foreign object such as hair is caught in the battery door when it is closed.
- After exposure to water, sweat or dust, the hearing aid is cleaned and dried.
- The hearing aid is used and maintained as described in this user guide.

① Use of your hearing aid around water can restrict air flow to the batteries causing it to stop working. Should your hearing aid stop working after coming into contact with water, refer to the troubleshooting steps in chapter 20.

Daily

Inspect the earpiece for earwax and moisture deposit. Clean the surfaces with a lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your hearing aid. It is not recommended to rinse with water. If you need to clean your hearing aid intensively, ask your hearing care professional for advice and information on filters or drying capsules.

Weekly

Clean the earpiece with a soft, damp cloth or with a special cleaning cloth for hearing aids. For more in depth maintenance instructions or for more than basic cleaning, please see your hearing care professional.

16. Exchanging the earwax protection

Your hearing aid is equipped with an earwax protection to protect the speaker from earwax damage.


Please check the earwax protector regularly and exchange it if it looks dirty or if the hearing aid volume or sound quality is reduced. The earwax protection should be replaced every four to eight weeks.

To replace the earwax protector on the receiver, only use the wax filter tool that was recommended by your hearing care professional (see page 3). Consult your hearing care professional about the availability of the wax filter tools.

16.1 Removing the earpiece from the speaker


1.

Remove the earpiece from the speaker by holding the speaker in one hand and the earpiece in the other.

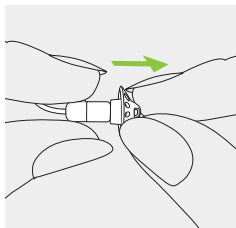


2.

Gently pull off the earpiece to remove.

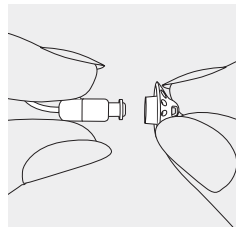

3.

Clean the speaker with a lint-free cloth.

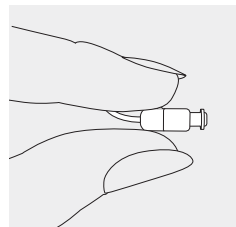


16.2 Exchanging the earwax protection with CeruStop

To replace the earwax protector on the receiver, only use the wax filter tool that was recommended by your hearing care professional (see page 3).


1.

Insert the removal side of the exchange tool into the used earwax protector. The shaft of the holder should touch the rim of the earwax protector.

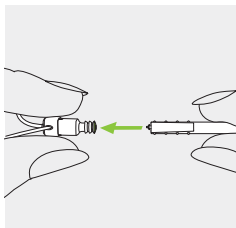


2.

Carefully pull and remove the earwax protector straight out of the speaker. Do not twist the earwax protector when removing.

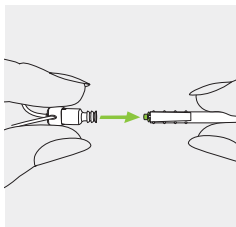


3.

To insert the new earwax protector, gently push the insertion side of the exchange tool straight into the hole of the speaker until the outer ring is perfectly aligned.

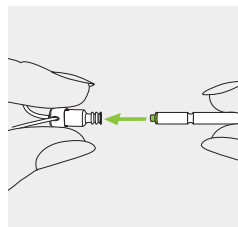

4.


Pull the tool straight out. The new earwax protector will remain in place. Now attach your earpiece to the speaker.

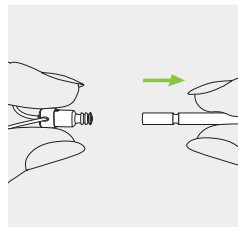

16.3 Exchanging the earwax protection with CeruShield™ Disk


1.

Gently rotate the CeruShield™ Disk in the direction indicated by the arrows until a free opening appears under the bin icon.

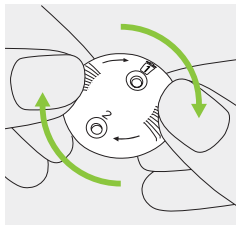


2.

To remove a used earwax protector carefully insert the speaker into the free opening (under bin icon) until it clicks. The used earwax protector will remain in the disk when you lift the speaker away.

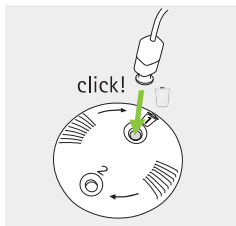


3.

To insert a new earwax protector carefully insert the speaker into the opening where the new earwax protector is visible (opposite the bin icon) until it clicks. Lift the speaker away from the disk, the new earwax protector is now positioned inside the speaker.

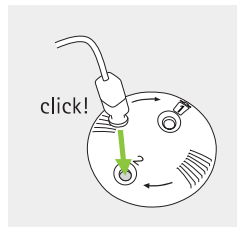


16.4 Attaching the earpiece to the speaker


1.
Hold the speaker in one hand
and the earpiece in the other.

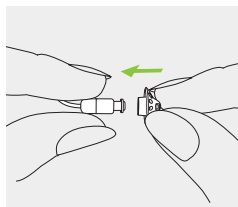

2.
Slide the earpiece over the
speaker sound outlet.

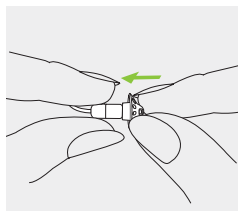

3.
The speaker and the earpiece
should fit perfectly together.

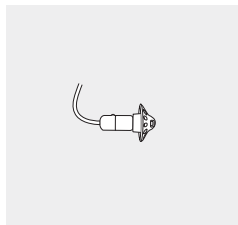

17. Service and warranty

Local warranty

Please ask the hearing care professional where you purchased your hearing aids about the terms of the local warranty.

International warranty

Sonova AG offers a one year limited international warranty, valid from the date of purchase. This limited warranty covers manufacturing and material defects in the hearing aid, but not accessories such as batteries, tubes, earpieces, external receivers. The warranty only comes into force if a proof of purchase is shown.

The international warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods.

Warranty limitation

This warranty does not cover damage from improper handling or care, exposure to chemicals or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void. This warranty does not include any services performed by a hearing care professional in their office.

Serial number (left side):

Serial number (right side):

Date of purchase:

Authorized hearing care professional (stamp/signature):

18. Compliance information

Europe:

Declaration of Conformity for the hearing aid

Hereby Sonova AG declares that this product meets the requirements of the Medical Devices Regulation (EU) 2017/745 as well as the Radio Equipment Directive 2014/53/EU.

The full text of the EU Declaration of Conformity can be obtained from the manufacturer or the local Phonak representative whose address can be taken from the list on www.phonak.com/en-int/certificates (Phonak worldwide locations).

Australia/New Zealand:



Indicates a device's compliance with applicable Radio Spectrum Management's (RSM) and Australian Communications and Media Authority (ACMA) regulatory arrangements for the legal sale in New Zealand and Australia. The compliance label R-NZ is for radio products supplied in the New Zealand market under conformity level A1.

The wireless models listed on page 2 are certified under:

Phonak Terra RIC-312

Phonak Terra+ RIC-312

USA	FCC ID: KWC-MZP
Canada	IC: 2262A-MZP

Notice 1:

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

Notice 2:

Changes or modifications made to this device not expressly approved by Sonova AG may void the FCC authorization to operate this device.

Notice 3:

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and ICES-003 of Industry Canada. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is

no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Radio information of your wireless hearing aid

Antenna type	Resonant loop antenna
Operation frequency	2.4 GHz – 2.48 GHz
Modulation	GFSK, Pi/4 DPSK, GMSK
Radiated power (EIRP)	<2.5 mW

Bluetooth®

Range	~1m
Bluetooth®	4.2 Dual-Mode
Profiles supported	HFP (Hands-free profile), A2DP

Electromagnetic compatibility (EMC)

Phonak hearing aids are EMC-tested in conformity with the requirements of IEC 60601-1-2:2014 or IEC 60601-1-2:2014 A1 2020.

19. Information and description of symbols



With the CE symbol, Sonova AG confirms that this product – including accessories – meets the requirements of the Medical Devices Regulation (EU) 2017/745 as well as the Radio Equipment Directive 2014/53/EU. The numbers after the CE symbol correspond to the code of certified institutions that were consulted under the above-mentioned regulation and directive.



This symbol indicates that the products described in these user instructions adhere to the requirements for an applied part of Type B of EN 60601-1. The surface of the hearing aid is specified as an applied part of Type B.



Indicates the medical device manufacturer, as defined in Medical Device Regulation (EU) 2017/745.



Indicates the date when the medical device was manufactured.



Indicates the Authorized representative in the European Community. The EC REP is also the importer to the European Union.



This symbol indicates that it is important for the user to read and take into account the relevant information in these user guides.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices in these user guides.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices related to batteries in these user guides.



Important information for handling and product safety.



This symbol confers that the electromagnetic interference from the device is under limits approved by the US Federal Communications Commission.



Indicates a device's compliance with applicable Radio Spectrum Management's (RSM) and Australian Communications and Media Authority (ACMA) regulatory arrangements for the legal sale in New Zealand and Australia.



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Sonova AG is under license. Other trademarks and trade names are those of their respective owners.



Japanese mark for certified radio equipment.



Indicates the manufacturer's serial number so that a specific medical device can be identified.



Indicates the manufacturer's catalogue number so that the medical device can be identified.



Indicates that the device is a medical device.



This symbol indicates that electronic instructions for use are available.



This symbol indicates a device that is not safe to use in magnetic resonance (MR) environment (e.g. during an MRI scan).

IP68

Ingress Protection Rating. IP68 rating indicates that the hearing aid is water and dust resistant. It survived continuous immersion in 1 meter of fresh water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard.



Indicates the temperature limits to which the medical device can be safely exposed.



Indicates the range of humidity to which the medical device can be safely exposed.



Indicates the range of atmospheric pressure to which the medical device can be safely exposed.



During transportation keep dry.



The symbol with the crossed-out garbage bin is to make you aware that this hearing aid as well as batteries may not be thrown away as normal household waste. Please dispose of old or unused hearing aids, at waste disposal sites intended for electronic waste, or give your hearing aid to your hearing care professional for disposal. Proper disposal protects the environment and health.

20. Troubleshooting

Cause	Possible remedy
Issue: Hearing aid whistles	
Hearing aid not inserted correctly into ear	Insert hearing aid correctly (chapter 6)
Cerumen in ear canal	Contact your hearing care professional
Issue: Hearing aid too loud	
Volume too loud	Decrease volume if volume control available (chapter 8)
Issue: Hearing aid not loud enough or distorted	
Volume too low	Increase volume if volume control available (chapter 8)
Low battery	Change battery (chapter 5)
Earpiece/speaker blocked	Clean speaker opening/earpiece (chapter 16)
Hearing has changed	Contact your hearing care professional
Issue: Hearing aid plays two beeps	
Indication for low battery	Change battery within the next 30 minutes (chapter 5)

Cause	Possible remedy
Issue: Hearing aid not functioning (no amplification)	
Earpiece/speaker blocked	Clean speaker opening/earpiece (chapter 16)
Hearing aid switched off	Press the lower part of the multi-function button for 3 seconds (chapter 8)
Battery fully discharged	Change battery (chapter 5)
Battery not inserted correctly	Insert battery correctly (chapter 5)
Issue: Hearing aid will not switch on	
Battery fully discharged	Change battery (chapter 5)
Issue: Phone call function does not work	
Hearing aid in flight mode	Switch hearing aid off and then on again (chapter 12.2)
Hearing aid not paired to the phone	Pair it to the phone (chapter 10)
Issue: Hearing aid switches on & off (intermittent)	
Moisture on battery or hearing aid	Wipe battery and hearing aid with dry cloth

❗ If the problem persists, contact your hearing care professional for assistance.

Please check <https://www.phonak.com/en-int/support> for further information.

21. Important safety information

Please read the relevant safety information and limitations-of-use information on the following pages before using your hearing aid.

Intended use

The hearing aid is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing

Intended user

Intended for:

- People with hearing loss
- Caregivers of the person with hearing loss
- Hearing care professional responsible for adjusting the hearing aid

Intended medical indication

Clinical indication for the use of hearing aids is:
Presence of hearing loss:

- Unilateral or bilateral
- Conductive, sensorineural or mixed
- Mild to profound

Medical contraindications

Medical contraindications for the use of hearing aids are:

- Deformity of the ear (i.e. closed ear canal; absence of the auricle)
- Neural hearing loss (i.e. retro-cochlear pathologies such as absent/ non-viable auditory nerve)

Intended patient population

This device is intended for patients from 8 years of age who fulfil the clinical indication for this product.

Clinical benefit

Improvement in speech understanding.

Side effects

Physiological side-effects of hearing aids like tinnitus, dizziness, wax build up, too much pressure, sweating or moisture, blisters, itching and/or rashes, plugged or fullness and their consequences like headache and/or ear pain, may be resolved or reduced by your hearing care professional. Conventional hearing aids have the potential to expose patients to higher levels of sound exposure, which might result in threshold shifts in the frequency range affected by acoustic trauma.

The primary criteria for the referral of a patient for a medical or other specialist opinion and / or treatment are as follows:

- Visible congenital or traumatic deformity of the ear;
- History of active drainage from the ear in the previous 90 days;
- History of sudden or rapidly progressive hearing loss in one or both ears within the previous 90 days;
- Acute or chronic dizziness;

- Audiometric air-bone gap equal to or greater than 15 dB at 500 Hz, 1000 Hz and 2000 Hz;
- Visible evidence of significant cerumen accumulation or a foreign body in the ear canal;
- Pain or discomfort in the ear;
- Abnormal appearance of the eardrum and ear canal such as:
 - Inflammation of the external auditory canal
 - Perforated eardrum
 - Other abnormalities which the hearing care professional believes are of medical concern

The hearing care professional may decide that referral is not appropriate or in the best interests of the patient when the following applies:

- When there is sufficient evidence that the condition has been fully investigated by a medical specialist and any possible treatment has been provided;
- The condition has not worsened or changed significantly since the previous investigation and / or treatment.

If the patient has given their informed and competent decision not to accept advice to seek a medical opinion, it is permissible to proceed to recommend appropriate hearing aid systems subject to the following considerations:

- The recommendation will not have any adverse effects on the patient's health or general well-being;
- The records confirm that all necessary considerations about the patient's best interests have been made.

If legally required, the patient has signed a disclaimer to confirm that the referral advice has not been accepted and that it is an informed decision.

The hearing aid is suitable for the home healthcare environment and due to their portability it may happen that it is used in professional healthcare facility environment like physician offices, dental offices etc.

A hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Infrequent use of a hearing aid does not permit a user to attain full benefit from it. The use of a hearing aid is only part of hearing habilitation and may need to be supplemented by auditory training and instruction in lipreading.






Feedback, poor sound quality, too loud or too soft sounds, inadequate fitting or problems when chewing or swallowing, can be resolved or improved during the fine-tuning process at the fitting procedure by your hearing care professional.


Any serious incident that has occurred in relation to the hearing aid, should be reported to the manufacturer representative and the competent authority of the state of residence. The serious incident is described as any incident that directly or indirectly led, might have led or might lead to any of the following:


- a) the death of a patient, user or other person
- b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health
- c) a serious public health threat



To report an unexpected operation or event, please contact the manufacturer or a representative.


23.1 Hazard warnings

-  Your hearing aids operate between 2.4 GHz–2.48 GHz frequency range. When flying please check if flight operator requires devices to be switched into flight mode, see chapter 12.
-  The hearing aids (specially programmed for each hearing loss) must only be used by the intended person. They should not be used by any other person as they could damage hearing.
-  Changes or modifications to the hearing aid that were not explicitly approved by Sonova AG are not permitted. Such changes may damage your ear or the hearing aid.
-  Do not use the hearing aids in explosive areas (mines or industrial areas with danger of explosions, oxygen rich environments or areas where flammable anesthetics are handled). The device is not ATEX certified.
-  If you feel pain in or behind your ear, if it is inflamed or if skin irritation and accelerated accumulations of earwax occur, please check with your hearing care professional or physician.


 Hearing programs in the directional microphone mode reduce background noises. Please be aware that warning signals or noises coming from behind, e. g. cars, are partially or entirely suppressed.

 This hearing aid is not for children below 36 months. The usage of this device by children and individuals with cognitive impairment should be supervised at all times to ensure their safety. The hearing aid is a small device and contains small parts. Do not leave children and individuals with cognitive impairment unsupervised with this hearing aid. If swallowed, consult a physician or hospital immediately as the hearing aid or its parts can cause choking!

  **WARNING:** Device batteries are hazardous and can cause serious injuries if they are swallowed or placed inside any part of the body, whether the battery is used or new! Keep out of the reach of children, individuals with cognitive impairment, or pets. If you suspect that a battery was swallowed or placed inside any part of the body, immediately consult your physician without any delay!

 The following is only applicable for persons with active implantable medical devices (i.e. pacemakers, defibrillators, etc.):

- Keep the hearing aid at least 15 cm (6 inches) away from the active implant. If you experience any interference, do not use the hearing aids and contact the manufacturer of the active implant. Please, note that interference can also be caused by power lines, electrostatic discharge, airport metal detectors etc.
- Keep magnets (i.e. battery handling tool, EasyPhone magnet, etc.) at least 15 cm (6 inches) away from the active implant.

 Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

⚠ Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the hearing aid or accessory, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

⚠ Avoid strong physical impacts to the ear when wearing a hearing aid with customized earpiece. The stability of customized earpieces is designed for normal use. A strong physical impact to the ear (e.g. during sports) may cause the customized earpiece to break. This may lead to perforation of the ear canal or eardrum.

⚠ After mechanical stress or shock to the hearing aid, please ensure the hearing aid shell is intact before placing it in the ear.

⚠ Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

⚠ The majority of hearing aid users have a hearing loss for which a deterioration in hearing is not to be expected with regular use of a hearing aid in everyday situations. Only a small group of hearing aid wearers with hearing loss, may be at risk of deterioration in hearing after a long period of use.

⚠ Hearing aids should not be fitted with domes / wax protection systems when used by patients with perforated eardrums, inflamed ear canals or otherwise exposed middle ear cavities. In these cases, we recommend the use of a classic earmold. In the unlikely case that any part of this product should remain in the ear canal, it is strongly recommended to see a physician for safe removal.

⚠ In very rare cases if not securely attached, an eartip or part of it can remain in your ear canal when removing the hearing aid from the ear. In the unlikely case that such part gets stuck in your ear canal, it is strongly recommended to see a physician for safe removal.

⚠ Do not remove receiver cables from the hearing instruments. If removal or exchange is required it is recommended to contact the hearing care professional.

⚠ Titanium shells of custom hearing aids and earpieces can contain small quantities of acrylates from adhesives. People who are sensitive to such substances may potentially experience an allergic skin reaction. Please stop using the device immediately if such a reaction occurs and check with your hearing care professional or physician.

⚠ Patients implanted with magnetic externally programmable CSF shunt valves may have a risk of experiencing an unintended change in their valve setting when exposed to strong magnetic fields.

The receiver (loudspeaker) of hearing aids, the Lyric SoundLync, the Lyric programming wand and the MiniControl contain static magnets. Keep a distance of 2 inches (approx. 5 cm) or more between the magnets and the site of the implanted shunt valve.

⚠ Special medical or dental examination including radiation described below, may adversely affect the correct functioning of your hearing aids. Remove and keep them outside the examination room/area before undergoing:

- Medical or dental examination with X-ray (also CT scan).
- Medical examinations with MRI/NMRI scans, generating magnetic fields.

Hearing aids don't need to be removed when passing security gates (airports etc.). If X-ray is used at all, it will be in very low doses, and will not affect the hearing aids.

23.2 Information on product safety

- ① These hearing aids are water resistant and not waterproof. They are designed to withstand normal activities and occasional accidental exposure to extreme conditions. Never immerse your hearing aid in water! These hearing aids are not specifically designed for extended periods of water submersion on a continual basis, that is worn in activities such as swimming or bathing. Always remove your hearing aid before these activities, as the hearing aid contains sensitive electronic parts.
- ① Never wash the microphone inputs. Doing so could cause it to lose its special acoustic features.
- ① Protect your hearing aid from heat and sunlight (never leave near a window or in the car). Never use a microwave or other heating devices to dry your hearing aid. Ask your hearing care professional about suitable drying methods.
- ① When you are not using your hearing aid, leave the battery door open so that any moisture can evaporate. Make sure that you always completely dry your hearing aid after use. Store the hearing aid in a safe, dry and clean place.
- ① Do not drop your hearing aid! Dropping onto a hard surface can damage your hearing aid.
- ① Always use new batteries for your hearing aid. In case a battery is leaking, replace it immediately with a new one to avoid any skin irritation. You can return used batteries to your hearing care professional.
- ① Remove the battery if you are not using your hearing aid for a long period of time.
- ① If you are not using your hearing aid for a long period of time, store it in a box with a drying capsule or in a well ventilated location. This allows moisture to evaporate from your hearing aid and prevents a potential impact on its performance.

Your hearing care professional:



EC REP

Sonova Deutschland GmbH
Max-Eyth-Strasse 20
70736 Fellbach-Oeffingen
Germany

Australian Sponsor:

Sonova Australia Pty Ltd
12 Inglewood Place,
Norwest NSW 2153
Australia



Manufacturer:

Sonova AG
Laubisrütistrasse 28
CH-8712 Stäfa
Switzerland
www.phonak.com



7 6 13389 698268

029-1333-02/V1.00/2023-07/PM © 2023 Sonova AG All rights reserved

sonova
HEAR THE WORLD



CE
0459



Type: PDL
Document No.: PDL-16730
Rev.: 2
Title: 029-1333-02 User Guide Phonak Terra/Terra+ RIC-312 EN
Process: Innovation Management
Owner: 361OWIELGOSZ Oliwia Wielgosz

Status: CURRENT
Effective Date: 13-Nov-2023

Attributes

<u>Attribute Type</u>	<u>Value</u>	<u>Description</u>
Affected Site	1100	Stafa (CH)
Project	Prince RIC 22 extension	
	DEVELOPMENT	
Project Phase	Design Output	

Approvals

<u>Level</u>	<u>Actor</u>	<u>Job Title</u>	<u>Sign-off Date</u>	<u>Sign-off By</u>
1	Diana Pena-Waldmann	Quality Manager	09-Nov-2023	11DPENAWALDM
2	Bettina Joss	Product Manager	10-Nov-2023	11BJOSS
2	Stefanie Hensler	Clinical Affairs Manager	13-Nov-2023	11SHENSLER
2	David Sooprayen	Regulatory Affairs Manager	10-Nov-2023	22DSOOPRAYEN
2	Kevin Stehrenberger	Usability Engineer	10-Nov-2023	11KSTEHRENBE
2	Luc Nimeskern	Design Quality Assurance Manager	13-Nov-2023	11LNIMESKERN

Revision Notes

<u>Access Activity</u>	<u>Note</u>	<u>Accessed By</u>	<u>Accessed Date</u>
Remark	1. Chapters "Exchanging the earwax protection with CeruStop" and "Exchanging the earwax protection with Cerushield" added, 2. RMR alignment, GPD alignment, 3. name on the cover changed to Phonak Terra (TM) / Phonak Terra(TM)+ RIC, 4. connectivity graphic changed 5. "Electromagnetic emission" table removed	361OWIELGOSZ	08-Nov-2023